POWERED BY NOVAMIN®

Calcium Sodium Phosphosilicate cleaning and desensitization powder

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Sylc® is a registered trademark of Denfotex Research, Ltd.
Novamin® is a registered trademark of
GlaxoSmithKline plc.
© 2013 Denfotex Research Ltd. All rights reserved.

23-1001 Rev.1 01/2013/ENGLISH / GERMAN / FRENCH /
ITALIAN / SPANISH

DIRECTIONS FOR USE

CAUTION: Federal law restricts this device to be used by or on the order of a licensed dentist.

DESCRIPTION: Sylc® is designed to provide a prophylaxis treatment for dental enamel and to physically occlude dentin tubules for the management of sensitive teeth. Sylc® is a dry powder (calcium sodium phosphosilicate), composed of elements that occur naturally in the body's hard tissues (Ca, Na, Si, P, and O). When exposed to an aqueous environment, Sylc® undergoes a rapid surface reaction, allowing it to physically adhere to exposed dentin and to physically occlude tubules. Within a short period of time, essentially all of the Sylc® reacts to form hydroxycarbonate apatite (HCA), which is chemically similar to natural tooth mineral. Sylc® removes plaque, soft deposits, and surface stains. It is also recommended for the following procedures:

- Removal of extrinsic stains, e.g. coffee, tea and tobacco
- Treatment of hypersensitivity
- Surface preparation prior to bonding and sealants
- Before and after scaling and root planing procedure to reduce sensitivity
- Prophylaxis of orthodontic patients
- Plaque removal prior to fluoride treatments
- Pre and post whitening treatments

INDICATIONS: Sylc® is intended for cleaning and polishing procedures as part of a dental prophylaxis treatment. The powder is also intended to provide relief from tooth sensitivity during the prophylaxis treatment. Sylc® is intended to be used in Denfotex Research recommended air polishing equipment or handheld devices.

INGREDIENTS: Sylc® particulate (calcium sodium phosphosilicate).

CONTRAINDICATIONS: Sylc® should not be considered for patients with a known allergy to silica.

PRECAUTIONS:

- During the preparation process and application, it is recommended that both clinician and patient wear protective eyewear.
- If any material gets into the eyes, immediately flush with water.
- Delivering Sylc® with air polishing equipment at high air pressures (psi), may cause injury to the gum tissues or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended not to direct Sylc® at the gum tissue or into the gingival sulcus.
- Avoid directing Sylc® powder toward restorations, crowns and bridgework as this could damage the restoration.
- Avoid directing Sylc® powder on orthodontic brackets as this could dull the metal.
- Empty the powder chamber daily, especially in areas of the country with high humidity to avoid clogging.
- Prophy tip lifetime will be optimized when working no higher than the recommended operating air pressure.
- Safety and effectiveness in pregnant women and children have not been established.

USAGE INSTRUCTIONS

1. Sylc® is intended to be used in Denfotex Research- recommended air polishing equipment or handheld devices.
2. Shake the bottle well before use. The Sylc® bottle is opened by turning the lid counterclockwise.
3. Pour Sylc® powder into the delivery system (chamber) at the maximum level recommended by equipment manufacturer.
4. Denfotex Research recommends Sylc® be applied at an air pressure range of 40-46 psi or approximately 2.8-3.2 bar. Equipment should be set to a minimum powder flow and powder uptake.
5. If increased cleaning effect is required, increase flow and powder settings accordingly.
6. Following treatment, the patient should not eat or drink for at least 60 minutes.
7. Sylc® can be used on patients who are on a sodium-restricted diet.

STABILITY

- Do not use if bottle or sachet is opened or damaged.
- Do not use after expiration date.
- Once bottle is open use the contents within 3 months
- Store in a DRY place at room temperature between 15-25 °C and 60-75 °F.

HOW SUPPLIED: Sylc® is supplied in bottles or unit dose

TECHNICAL CHANGES: Denfotex Research reserves the right to make changes to Sylc® usage instructions when required in the course of ongoing technical and scientific developments.

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