


VELOPEX[®]

Automatic X-ray Film Processor

Intra - X MK5

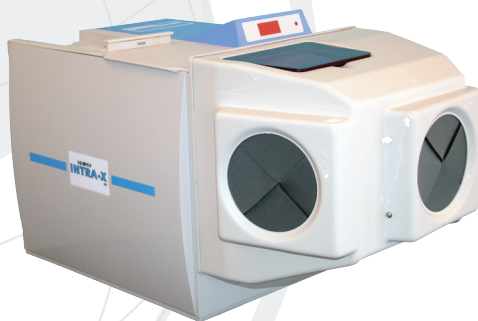

DAYLIGHT LOADER INSTALLATION



© February 2007 Medivance Instruments Limited
All rights reserved.
Issue 1 I/LIT9010M


Daylight Loader Installation



1.

Bring the Lid to the upright position. When operating, indication of the opened lid is displayed as: "Lid". Unscrew the retaining screw at the back of the machine above the Electric Plug and lift the Display Panel.

(Images for instruction ONLY)

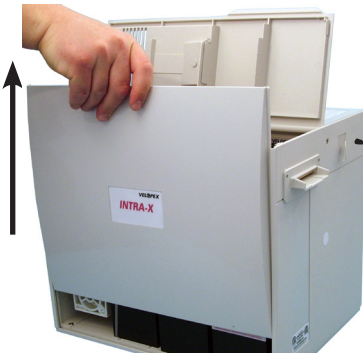

2.

Remove Side Panels by Sliding them Upwards in their Runners.



3a.

Remove All Packing / Wrapping Material and Modules.



TIP: Start with the Dryer Module, then the Water Module followed by the Fixer and Developer Modules. Those should be removed by sliding the Tanks out.

(Images for instruction ONLY)


Daylight Loader Installation (cont.)




3b.



WARNING: When Removing the Fixer and Developer modules, slide tanks sideways until completely clear of the immersion heaters mounted on the machine internal wall.

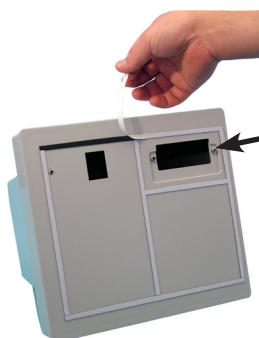

(Back of Loader)

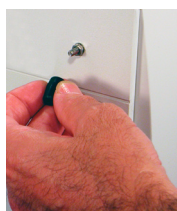

4.

Unscrew the Two Retaining Screws and Remove White protective strip from sealing material on the back of the loader. Remove Black Covers from the stud on the front panel.



(Front of Loader)

5.


Remove Protective Film from Viewing Window Cover.

Daylight Loader Installation (cont.)



- 6.** Slide Loader over Film Entry Guide and 'RUN' Sensor, Press the Loader firmly against Machine.

(Images for instruction ONLY)

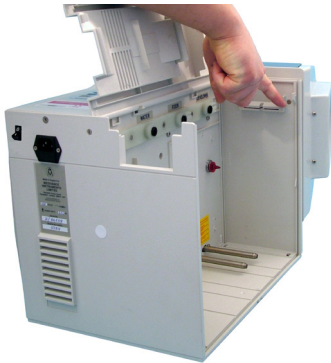

- 7.** Secure Loader with the Two Retaining Screws. Return side panels, lid panel and display-secure with retaining screw. Secure Loader with black knob screwed onto the Stud now positioned inside the Loader.

