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ONNEKSI OLKOON!

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SV

Disclaimer

The user shall determine the suitability of the Salli Saddle Chair® and its accessories for his or her intended use and shall assume all risk and liability in connection therewith; this will include all items manufactured or distributed by Easydoing Oy/Salli Systems ("Salli"). Easydoing cannot accept responsibility for injury and/or damage that occurs in connection with the testing, trial use or regular use of the product except as provided for in mandatory law. Great care should be taken when using the products and/or accessories related to product.

User is required to read all instruction manuals including these instructions of use before using the product and to follow such instructions when using the chair.


DO NOT USE THE PRODUCT UNDER ANY OF THE FOLLOWING CIRCUMSTANCES:

1. IF YOU GET ANY PELVIC, EXTREMITY, BACK, OR OTHER KINDS OF PAIN FROM USING THE CHAIR (OTHER THAN NORMAL MUSCLE SORENESS) AND THE PAIN DOES NOT PASS IN A FEW DAYS.
2. IF YOUR SENSE OF BALANCE OR MUSCLE CONTROL IS ABNORMAL OR IMPAIRED FOR ANY REASON.
3. IF YOU USE MEDICATION OR DRUGS THAT INTERFERE WITH YOUR SENSE OF BALANCE OR MUSCLE CONTROL.
4. IF YOU DO NOT OR CANNOT ADJUST YOUR WORKTOP TO A FUNCTIONAL HEIGHT THAT IS COMPARABLE WITH THE OPTIMAL HEIGHT OF THE SALLI SADDLE CHAIR®. THE DEFINITION OF THE OPTIMAL HEIGHT OF THE TABLE/WORK IS DESCRIBED IN THE FOLLOWING INSTRUCTIONS.

Due to the danger of falling down do not use yourself or allow children or anyone else to use a product unless the feet of the user reach the floor firmly when sitting down. For shorter users we recommend changing the gas cylinder to a shorter model to ensure the safe use of the product. A short gas cylinder is available from Salli and authorized dealers, and the gas cylinder can be changed by Salli facility or an authorized dealer.

ASSEMBLING SALLI

1. Place the chair base on floor, with the castors against the floor (Picture 1).
2. Put the wider end of the gas cylinder into the hole in the base (Picture 2).
3. Put the slimmer end into the barrel under the seat (Picture 3). You may need to pump the gas cylinder for a while in order to get it working properly.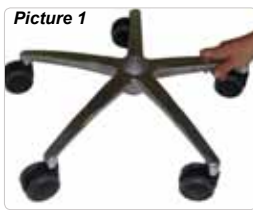

The chair is now ready.

SADDLE CHAIR ADJUSTMENT

Depending on the model, your Salli Saddle Chair is equipped with one or more of the following adjustments:

Professional / MultiAdjuster / Twin

As seen from behind the chair:


1. Height adjustment (all models)
2. Inclination adjustment *
3. Seat width adjustment (MultiAdjuster)

* Inclination adjustment: MultiAdjuster. Classic and Twin seats are available both with or without inclination adjustment. Salli Professional always without inclination.

Swing / SwingFit

As seen from behind the chair:

1. Height adjustment (all models)
2. Swinging mechanism (cannot be locked into any specific position)
3. Seat width adjustment (SwingFit)

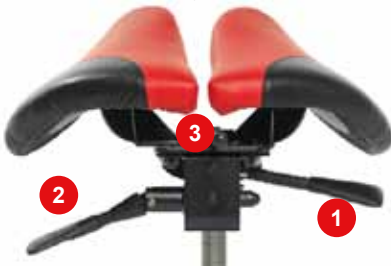INCLINATION ADJUSTMENT


1) Hold onto the chair with your hand and release the inclination mechanism by lifting up on the lever (Picture).

In the best inclination position you do not feel like sliding forward or backward, and sitting causes no pressure.

2) When the angle feels right, release the lever to lock the mechanism.

WIDTH ADJUSTMENT

1. Stand behind the chair. Release the seat width adjustment mechanism by hitting the lever briskly with your palm (Picture 1).
2. Move the seat parts into the desired position by sliding them together or apart (Picture 2).
3. Lock the mechanism by snapping the lever from right to left. You may need to repeat the procedure few times at the beginning, to find the optimal adjustment for you.

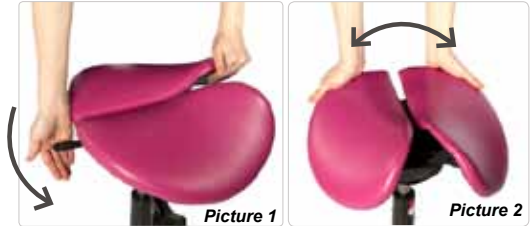We recommend that you try the inclination and width adjustments several times in different combinations to find the optimal adjustments for you.

SEAT WIDTH ADJUSTMENT PRINCIPLES

Optimal seat width adjustment varies with the user's individual physiology. We recommend keeping the middle gap as wide as comfortably possible. The wider the gap, the less pressure to the sensitive ducts under the pelvic floor.

LEATHER SEAT CARE AND CLEANING

The leather will stay clean longer if you wipe it every now and then with a moist cloth.

Remove stains immediately. Wipe lightly with a cloth or paper towel moistened with a mild soap and water solution.

You can also clean the seat with products made for leather furniture or riding equipment.

Note that bleeding textiles may colour light-coloured leather.

Marks on the leather and slight colour variations are natural to genuine leather and do not affect the durability of the product.

POLYURETHANE SEAT CARE AND CLEANING

The Salli polyurethane seat and the gas cylinder can be cleaned by wiping with disinfectant. Do not use a pressure washer.

GAS CYLINDER CARE AND LUBRICATION

If the gas cylinder becomes slow or stiff, spray a light coating of lubricant onto the thinner end of the gas cylinder.

CASTOR REPLACEMENT

1. Turn the chair sideways.
2. If you have strong hands, you can remove the castors by pulling. Otherwise you may need a lever, for example, a chisel or a screwdriver which you place between the castor and the base.
3. Push the new castors into place.

If a castor stem remains caught in the base, grip the stem with pliers and pull.

You may need to tap the pliers with a hammer until the stem comes loose.

If your castors get stiff, replace them all.

WEIGHT TOLERANCE

The recommended maximum weight for a Salli user is 120 kg (264 lbs), Salli Strong 150 kg (330 lbs).

DISASSEMBLING SALLI

We recommend using special tools for disassembling Salli. You can borrow the tools from your Salli dealer.

REMOVING THE SEAT

1. Set the gas cylinder in its longest position.
2. Turn the chair upside-down.
3. Place the semicircular tube of the disassembling tool at the junction of the gas cylinder and the seat, so that the wedge-shaped end points towards the base.
4. Lift the chair a little off the floor. This is easier if another person holds the chair up for you.
5. Hold the tool handle in position and hit the cantilever with a hammer until the seat separates from the gas cylinder.

REMOVING THE BASE

1. Ask someone to hold the chair up for you, seat downwards.
2. Position the disassembly bar tool at the bottom of the gas cylinder.
3. Hold the tool in position and hit the free end with a hammer a few times, until the cylinder separates from the base.

MAINTENANCE, RECYCLING AND DISPOSAL

The Salli Saddle Chair is designed for indoor use at normal room temperatures in a dry environment. For humid environments we recommend the rustproof Salli Stainless, as a normal gas cylinder will corrode if exposed to moisture.

To properly dispose of a Salli:

1. Remove the leather from the seat and discard.
2. Remove plastic and foam parts, including padding, control lever handle(s) and the mechanism's plastic parts, plastic castor components (the metal stem of the castor can be removed e.g. in bench vice with a hammer and a chisel). Dispose of plastic components in an appropriate way with other plastic garbage.
3. Recycle metal chair parts.

ACCESSORIES

Multiple accessories for different purposes are available for Salli Saddle Chairs.

Visit www.salli.com for more details.

EN SALLI'S POSITIVE EFFECTS ON YOUR BODY

Your sitting-related ailments have taken years or even decades to develop. The new ergonomics will start the positive change in your body immediately, but the deeper and wider impacts take time. For example, correcting your posture will take years. All the many positive effects of Salli Saddlechair are described on our webpage at www.salli.com.

SITTING DOWN ON SALLI

Before you sit down, make sure that the chair is not too high for you. The chair should be lower than your pelvis. If the chair is too high, lift up the lever and at the same time press down hard on the seat by your elbow or stomach. Once the seat is at the correct height, release the lever.

Stand beside the chair and lift one foot up over the back of the chair to the other side (Picture) and sit down. You may want to hold onto the chair with your hand while sitting down. This way the chair will not roll away from you and you will find the right sitting position easily.

WARNING!

Do not sit down on the chair the traditional way, i.e. from in front of the chair with your buttocks foremost. The chair might slide away and you might fall. You will also end up in the wrong sitting position on the front edge of the chair.

Make sure that you don't accidentally move the chair away when you lift your foot over it.

Salli Saddle Chairs are higher than traditional chairs.

CORRECT SITTING POSITION AND SEAT HEIGHT

In the optimal sitting position the angle between your thighs and upper body is about 135 degrees and your heels are positioned almost directly under your hip joints. In this position your pelvis will tilt forward so that your lower back forms a natural curve, and your upper back will straighten. Legs should be relaxed, not hanging in the air, and feet should rest on the floor. Thighs should be relaxed against the chair, forming a 90 degree angle relative to each other.

On the average, on a saddle chair one sits about 25–40 cm (10"–16") higher than customary. To find your optimal sitting position and seat height, do the following: Raise the seat so high that your feet hang in the air. Relax and let your thighs and pelvis stretch for 1–2 minutes. Let gravity do the stretching for you. Then slowly lower the chair with the lever until your heels touch the floor.

Picture 1 - Correct table height

TIPS FOR ADJUSTING TABLE HEIGHT

Adjust the table a bit higher for passive visual tasks (e.g. internet, reading) and a bit lower (3–4 cm) for active writing.

NOTE!

The sitting position on Salli Saddle Chair differs from the traditional chair. Instead of passively sitting in one position, the user's muscles and body work actively. This means that saddle chair sitting often demands some getting used to. Starting to use the Salli Saddle Chair is like starting a new sport – in the beginning your muscles and tissues can get sore. The sitting bones carry the weight of the upper body, which might cause your buttocks to feel sore in the beginning. Some people get used to the new sitting position in just one day, some take months to get used to it.

ASSORTED TIPS

- Remember to adjust the table and chair height according to the instructions above.
- Let your body get used to the new way of sitting: don't be alarmed by the discomfort and aches you might feel in the beginning.
- Learn to use the chair as a tool and learn to use your body while sitting.
- Remember to take breaks.
- Sit high and back enough.

After the introductory phase sitting on a correctly adjusted Salli is comfortable!

In the right position the buttocks and the back edge of the chair are level with each other. Sitting too far in the front diminishes both comfortability and sitting health.

TABLE HEIGHT

The standard table height is about 72 cm, but with Salli Saddle Chairs the table needs to be more than 80 cm high. We recommend an electronically adjustable table, see www.salli.com.

When you sit at the correct height, the optimal table height will be approximately at the height of your elbows (Picture 1). If your table cannot be adjusted to the proper height, you can raise it with our height adjustment tubes or by some other means.

A table positioned too high (Picture 2) or too low (Picture 3) can cause back, shoulder and neck problems. Your elbows should be relaxed and resting on table padding, with your arms at the sides of your body.

Picture 2 - Too high

Picture 3 - Too low

MOVING AROUND WITH SALLI

You can easily move sideways by pushing with one foot and pulling with the other (Picture). Place things you need around in such a way that you need to roll and reach to different heights; this is good from the point of view of sitting health, because moving activates muscles and metabolism. Instructions for easy and efficient exercises are available at Salli Systems and on www.salli.com.

Notice!

Hard floors and shoes with soft soles make moving around with Salli easy. Be careful not to lose control of the chair!

More tips on sitting and ergonomics
www.salli.com

Product Warranties

Salli Systems/Easydoing Oy ("Salli") gives (1) a standard warranty and (2) an extended warranty of ten years for Salli Saddle chairs to customers who register their chairs in Salli Systems database and respond to a customer feedback survey.

1. The standard warranty

The standard warranty from the date of purchase of the Salli product ("Product"), for any structural defect in the Product or a fault in its materials or assembly, which arise during regular and normal indoor use of the Product ("Defect"). The length of the warranty (2–5 years) for each individual Product is stated in the official product catalogue valid on the day of manufacture of the Product.

Any claim for Defect under this warranty must be made in writing to Salli immediately and in no case later than within seven (7) days from the date the distributor/end user became or should have become aware of the Defect. Any claims not made within said time-period will be null and void. The date of the purchase must be verified by the serial number of the Product and the receipt of the end user or book keeping of the distributor or Salli. The warranty time (2–5 years) is determined by the date of purchase.

The warranty given by Salli does not cover damage or faults caused by:

- any leather damaging liquids
- falling on the floor or dropping
- abrasion against hard, rough or sharp objects that leaves marks
- piercing or impressions made by sharp objects
- normal wear and tear
- pets
- any amendment or alteration made to the Product whatsoever by or at the request of distributor or end user
- any parts or objects other than those approved by Salli in writing prior to assembly used in the Product or in any way attached to the Product.

Distributor/end user shall send, at their cost, the defective Product to Salli's factory in Rautalampi, Finland or to Salli's warehouse or other place mutually agreed upon between the parties. Upon arrival of the defective Product, Salli shall inspect the Product without any delay.

Distributor/end user may also make a written description about the defected part and take a photo about it and send this material directly to Salli by mail or e-mail. Salli may also pay for the transportation of the defected part/product once it has verified that the defect is covered by the warranty. The repaired or replaced Product will be returned to the distributor or the end user by Salli. Should the Product not be defective or the Defect not be covered by this warranty, the distributor/end user shall reimburse to Salli reasonable costs for the inspection of the Product and possible shipments.

Should it be unreasonably expensive or impractical to send the defective Product to Salli factory or to clear the warranty coverage by written or picture materials, the Defect must be verified by an impartial expert who is agreed on by Salli and distributor/end user for his warranty to be valid.

When the warranty coverage is clear according to the fore said measures, Salli may also send the defected parts to the distributor/end user. The warranty of Salli does not cover the work or costs of assembly of the spare parts, or disassembly of the old parts, on behalf of the end user or distributor but will provide them with adequate written or oral instructions.

The leather quality in the upholstery of the Product may change according to the quality level. Quality A1 is practically faultless; A2 can have slightly unequal surfaces in different sizes and more normal liveliness of natural leather. A3 quality can have scars in the skin or even more different or unequal surface areas in the leather but no holes or areas with broken surface.

Salli's limited warranty is in lieu of all other warranties whether written, oral, express or implied. The distributor's/end user's sole claim is for repair or replacement, at Salli's sole discretion, of any defective Product covered by the warranty. Salli shall not be responsible for any indirect, consequential, incidental or punitive damages or lost profits or loss of goodwill. Salli does not make and hereby disclaims any other warranty, statutory or otherwise, express or implied with respect to the Product including, without limitation, any warranty of fitness for a particular purpose or merchantability. However, this Product Warranty does not and is not intended to limit the rights that consumers may have under mandatory law. Distributor shall be responsible for, and Salli shall have no obligation to honour, any warranties that Distributor or any of its sub distributors or any other third party, as the case may be, provides to its customers and/or end-users with respect to the Product that are in addition to, or inconsistent with, the warranties provided by Salli in this Agreement.

2. The extended warranty

The extended warranty is given for Salli Saddle chairs when a customer registers the Salli chair by using its individual product number in the database of Salli Systems, either on the internet (www.salli.com) or by posting a registration form. The extended warranty for the chair is ten (10) years under the conditions of use specified in Section 1. The extended warranty is only valid when the registered customer responds to a feedback survey six months after his/her registration, on the internet or by printing and posting the feedback form.

IDENTIFICATION OF ORIGINAL SALLI

All original Salli Saddle Chairs have an identification label with serial number. The plate can be found in the front panel of the seatbase.

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Käyttäjää päättää itse Salli Satulatuolin© ja lisävarusteiden sopivuudesta omaan käyttöönsä ja käyttää niitä omalla vastuullaan; tämä koskee kaikkia Easydoing Oy:n/Salli Systemsin ("Salli") valmistamia ja jakelemia tuotteita. Salli ei vastaa tuotteiden kokeilemisen, testikäytön tai säännöllisen käytön aikana aiheutuneista vammoista ja/tai vahingoista muilta osin kuin pakottavassa lainsäädännössä määrätään. Tuotteita ja niihin liittyviä lisävarusteita tulee käyttää suurta huolellisuutta noudattaen.