

SmartCartTM


THE MODULAR CART FOR THE DENTAL PERFECTIONIST

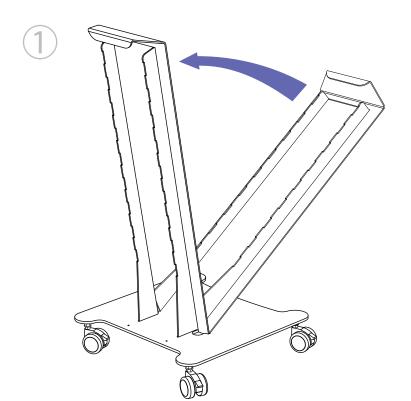


Lower the upright into the slots on the base & raise it into position.

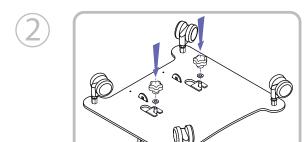


Place the washers & thumb-nuts onto their thread & tighten down.

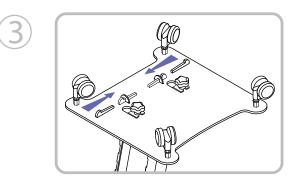


Securely knock each locking pin into place.

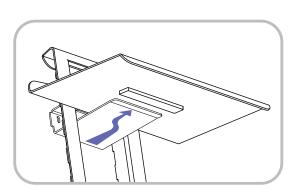


Raise the shelf & its support into place. Ensure its end slots into the pocket underneath the shelf

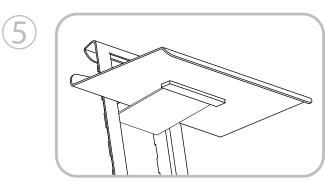

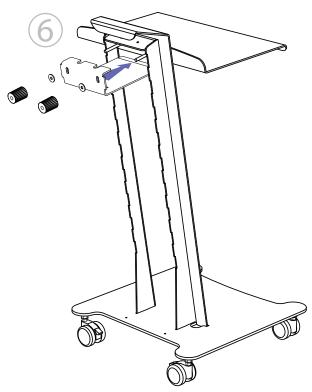

Screw up the thumb-nuts & secure the shelf.




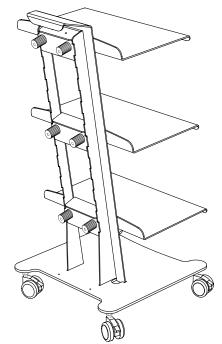

Repeat steps 4-6 for all shelves.

Parts list

- 1. SHELF THUMB NUT (6pcs)
- 2. WASHERS (6pcs)
- 3. SHELF SUPPORT (3pcs)
- 4. UPRIGHT (1pcs)
- 5. SHELF (3pcs)
- 6. BASE (1pcs)
- 7. LOCKING PIN (2pcs)
- 8. WASHER (2pcs)
- 9. BASE THUMB NUT (2pcs)

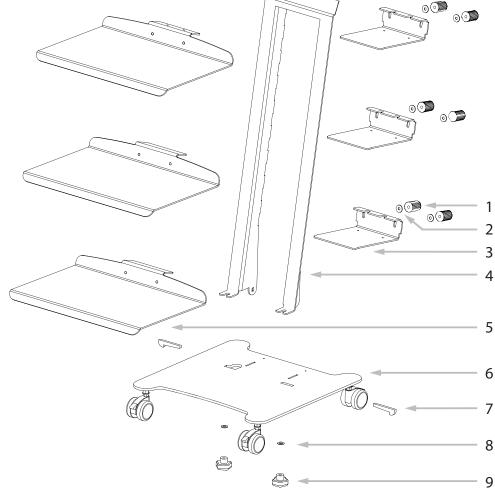



10Kg Maximum weight limit for individual shelves.

