
VISCHECK™

X-RAY QUALITY CONTROL SYSTEM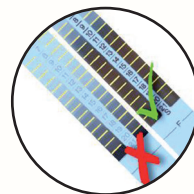

USER MANUAL

MEDIVANCE INSTRUMENTS LTD. Barretts Green Road London NW10 7AP United Kingdom

www.velopex.com

What is in the box

1. Visccheck Reader
2. Visccheck Reader Clip
3. Box of 20 Visccheck pre-exposed Visccheck films
- 3a. Visccheck film strip

1.

2.


3.

3a.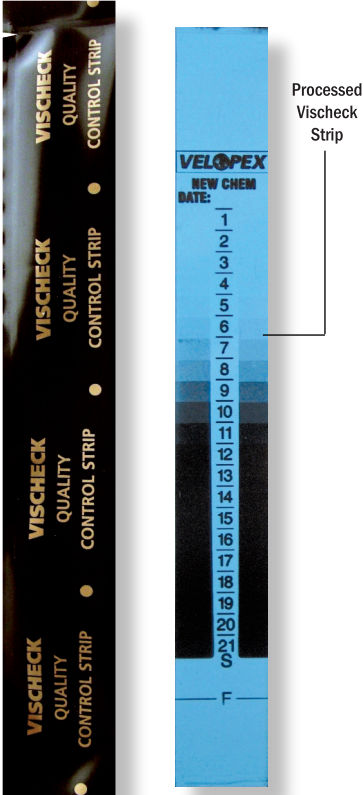

How it works

1. Process a Visccheck Strip making sure to open it in a darkroom or daylight loader. Post the strip into the processor as if it were an x-ray film.
2. Turn on the Visccheck Reader and insert the processed strip. The electronic Reader will then indicate the suitability of the processor to produce quality x-ray film.
3. Use the storage sleeve provided to keep the processed quality strip by dating it and keeping a record of quality management.

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How to use the X-Ray Quality Control System 1


Interpreting the Vischeck Reader 7

Changing the battery 11


Troubleshooting 13

Notes/Conditions 19

How to use the X-Ray Quality Control System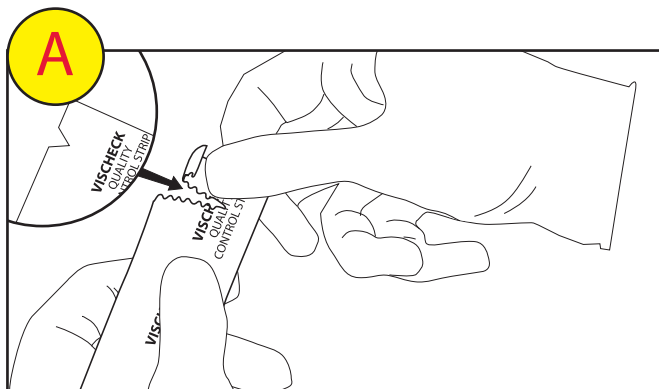

To open, tear across the black film pack at the small notch along its edge.

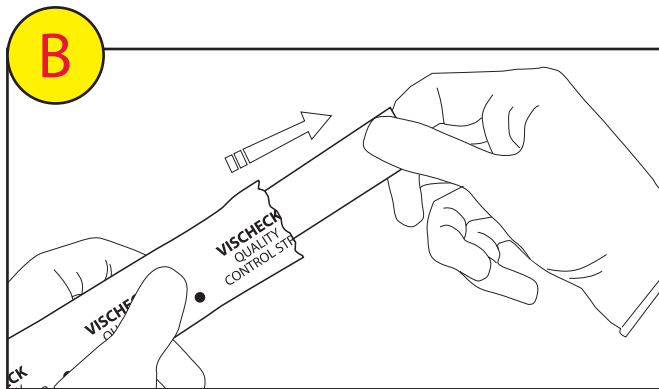Withdraw Vischeck Film from its pack and process it.

NOTE: Steps A and B must be done ONLY in a darkroom or in a daylight loader!

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
How to use the X-Ray Quality Control System (Cont.)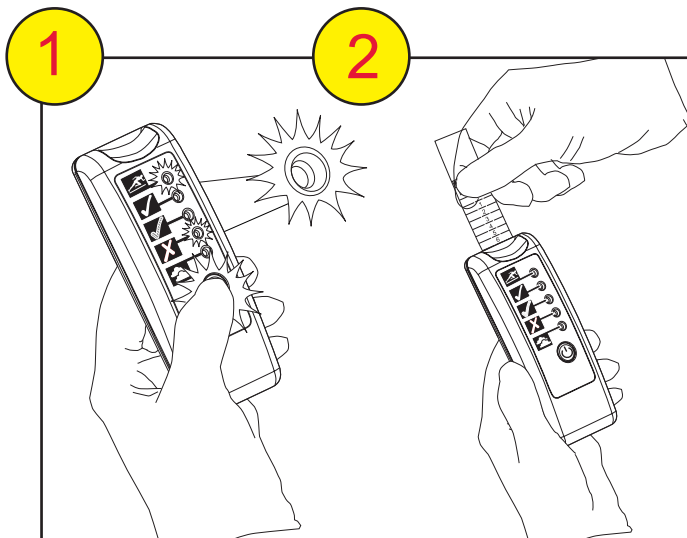

Reading the processed Visccheck strip:-

Press and hold the 'start' button on the Visccheck Reader.

The lights will flash from red to green, and when the Visccheck Reader is ready the green and red lights will both stay on.

Whilst green light is illuminated, insert processed Visccheck strip until strip stops naturally. Visccheck Reader will then indicate a condition.

NOTE: The Visccheck Reader turns off automatically after 20 seconds. Should this occur before the Visccheck strip is inserted, repress 'start' button and insert strip again.

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

E

Changing the battery

1. The Visccheck Reader will indicate when the battery is low by a flashing LED.
2. This LED will flash when the Visccheck Reader is switched on and will last approximately 3 seconds.
3. To replace batteries, open the battery compartment and replace with two AAA batteries.

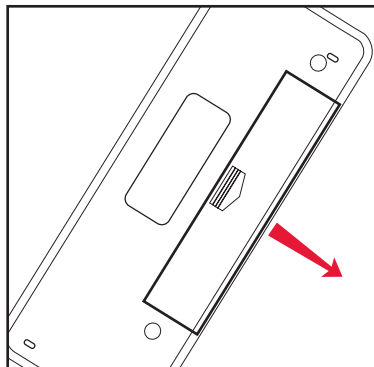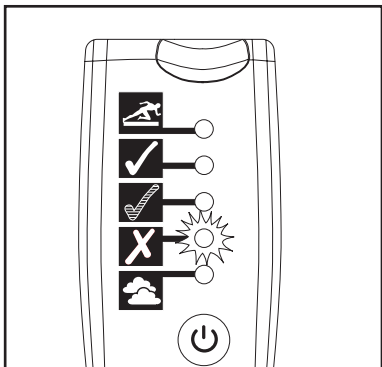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Troubleshooting

If Vischeck is indicating correctly but **x-ray film images** are unsatisfactory, check the following:

1. X-Ray film comes out blank

- (a.) x-ray unit not turned on - **make sure x-ray machinery is switched on**
- (b.) x-ray not exposing - **call technician**
 - **check processing chemical temperatures**

2. X-Ray film comes out too dark

- (a.) chemical temperature too high - **correct temperature level**
- (b.) chemical has been contaminated - **clean processor and replace chemical**
- (c.) film has been over-exposed - **check x-ray unit**

3. X-Ray film comes out too light

- (a.) chemical temperature too cold - **check temperature level**
- (b.) film is under-exposed - **check x-ray unit**
- (c.) error with film - **replace film**

4. X-Ray film is fogged

- (a.) film has been exposed to light - **check for light leaks**
- (b.) film is out of date - **replace film**
- (c.) film has been poorly stored - **store film in a cool area as instructed by manufacturer**

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Velopex is a registered trademark which covers all Velopex products.