

## AquaCare Stand: I/MAC 8516F (Single)

The AquaCare Stand has been designed to be compact, secure and mobile. The support shelf ensures the unit cannot slip when the stand is moved.

The stand enables the AquaCare unit to be moved conveniently into place when required.

### Dimensions:

Width: 35cm

Depth: 35cm

Height of Stand  
(Without Unit) : 68cm


(With Unit): 86cm

Weight: 3kg

The AquaCare Stand has a handle fitted to the top shelf to allow easy grip. This ensures stable and secure movement of the stand and AquaCare unit.

The AquaCare Stand comes fitted with an air filter. The height of the air filter can be adjusted to suit your needs.


The AquaCare Stand has wheels fitted to the base to allow ease of movement during use.

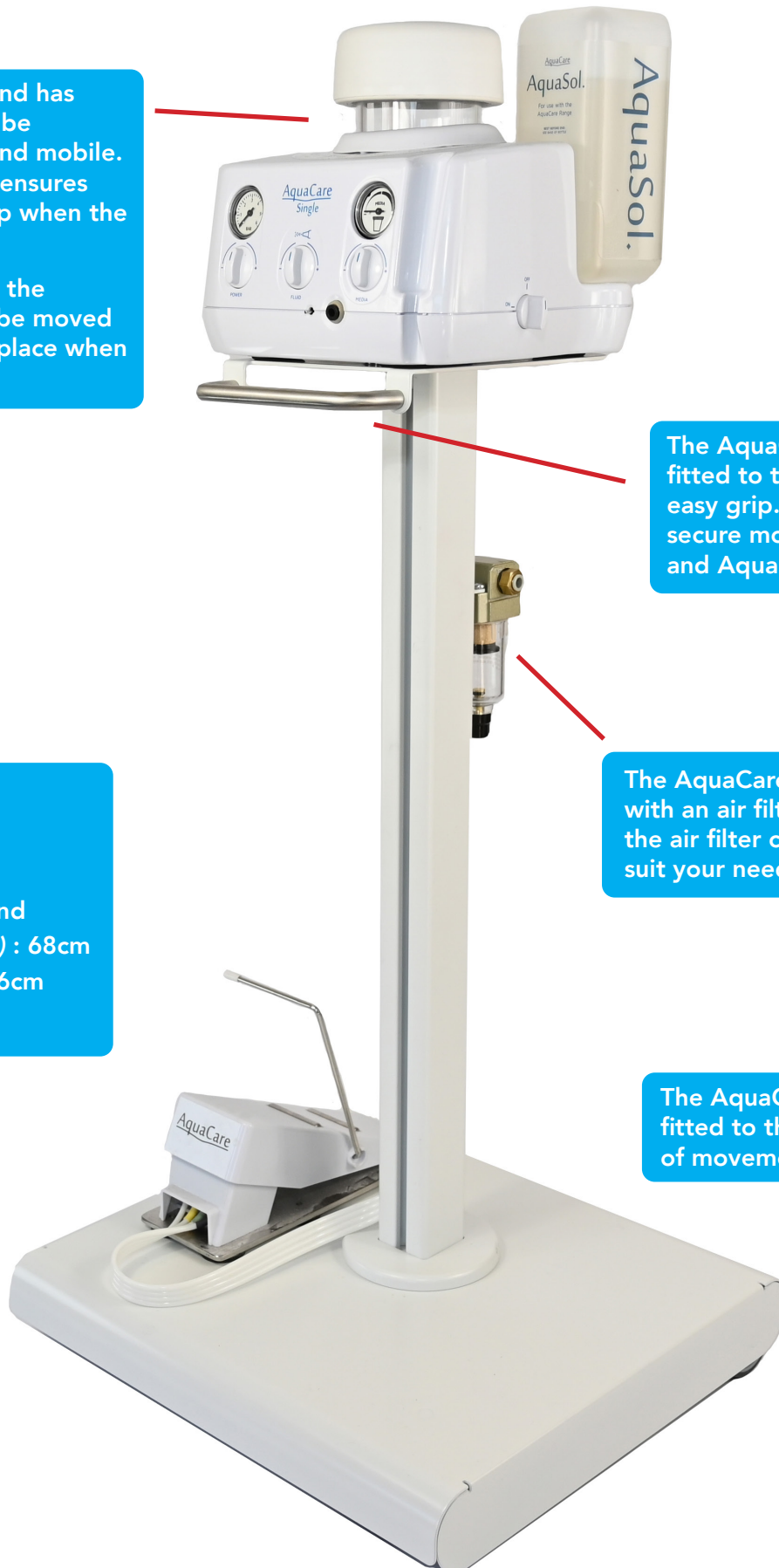

## Connecting the Foot Pedal

**NOTE:** Ensure the foot pedal tubing comes out the other end of the column support. Ensure that enough tubing is fed through to be able to connect to the machine properly.

**NOTE:** Insert the foot pedal tubing through the U shaped enclosure and push it up the column support.

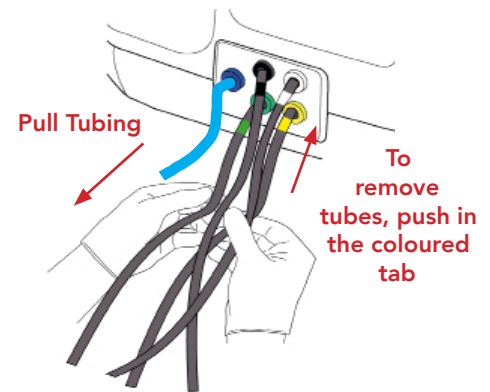


**NOTE:** To place the AquaCare onto the stand, first disconnect the air supply tubing and the foot pedal tubing from the device. Please make sure you push the coloured tabs in whilst pulling the tube out. Locate the device onto the shelf and then reconnect by pushing in the tubing according to the colour code shown.

## Assembly Instructions


### ONE NUT ASSEMBLY

The stand has been partially assembled and only requires the Metal Base to be attached to the column support with 1 screw and washer being placed through the base. Once fully assembled, situate the AquaCare unit on top of the support shelf placing it over the pins.

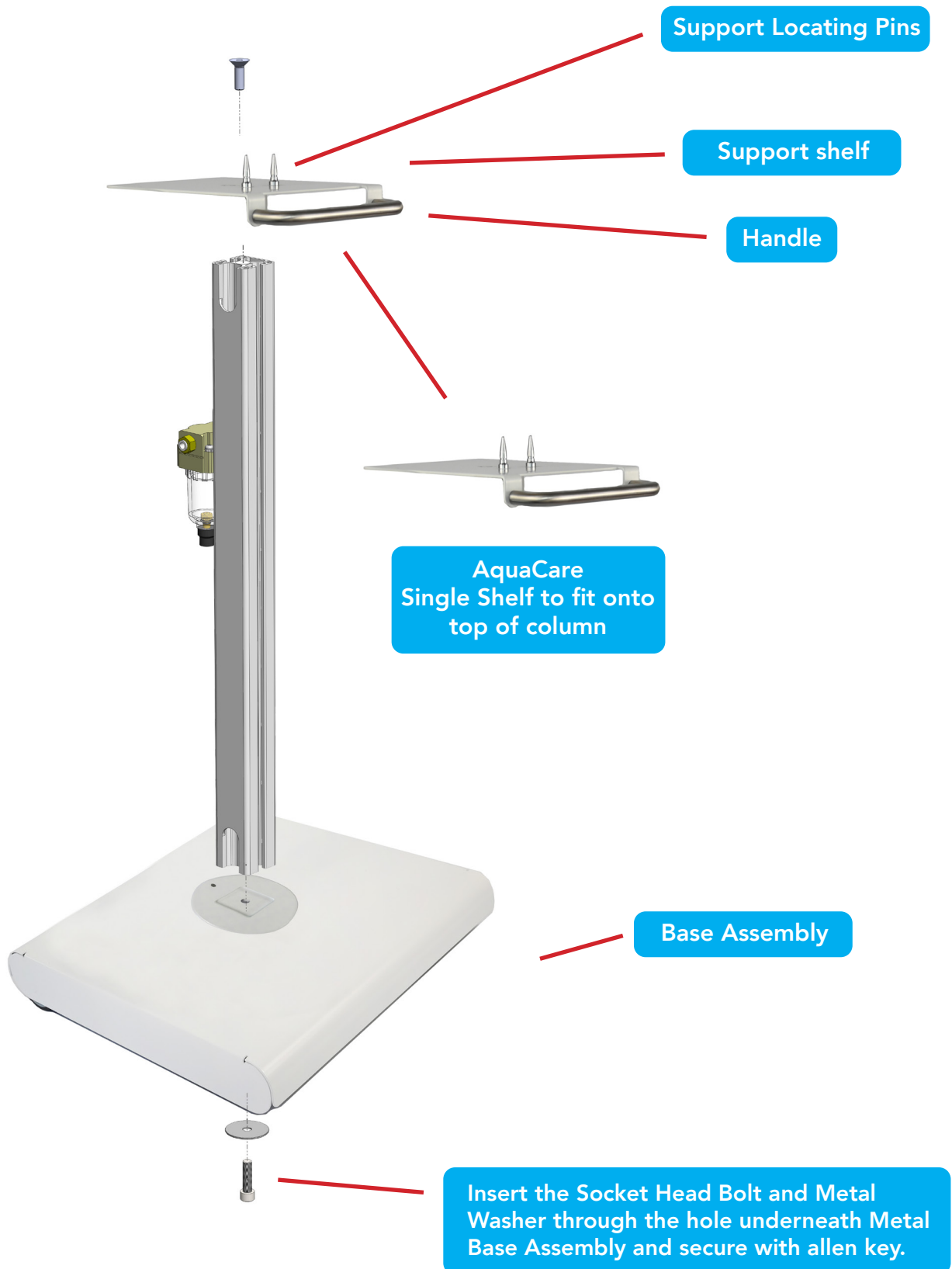

## Connecting the AquaCare unit

The supplied air filter removes moisture and oil out of the incoming air from the air supply system/compressor at the clinic. This helps to give an indication of the incoming air quality and ensure that better quality of air is supplied to the AquaCare unit.

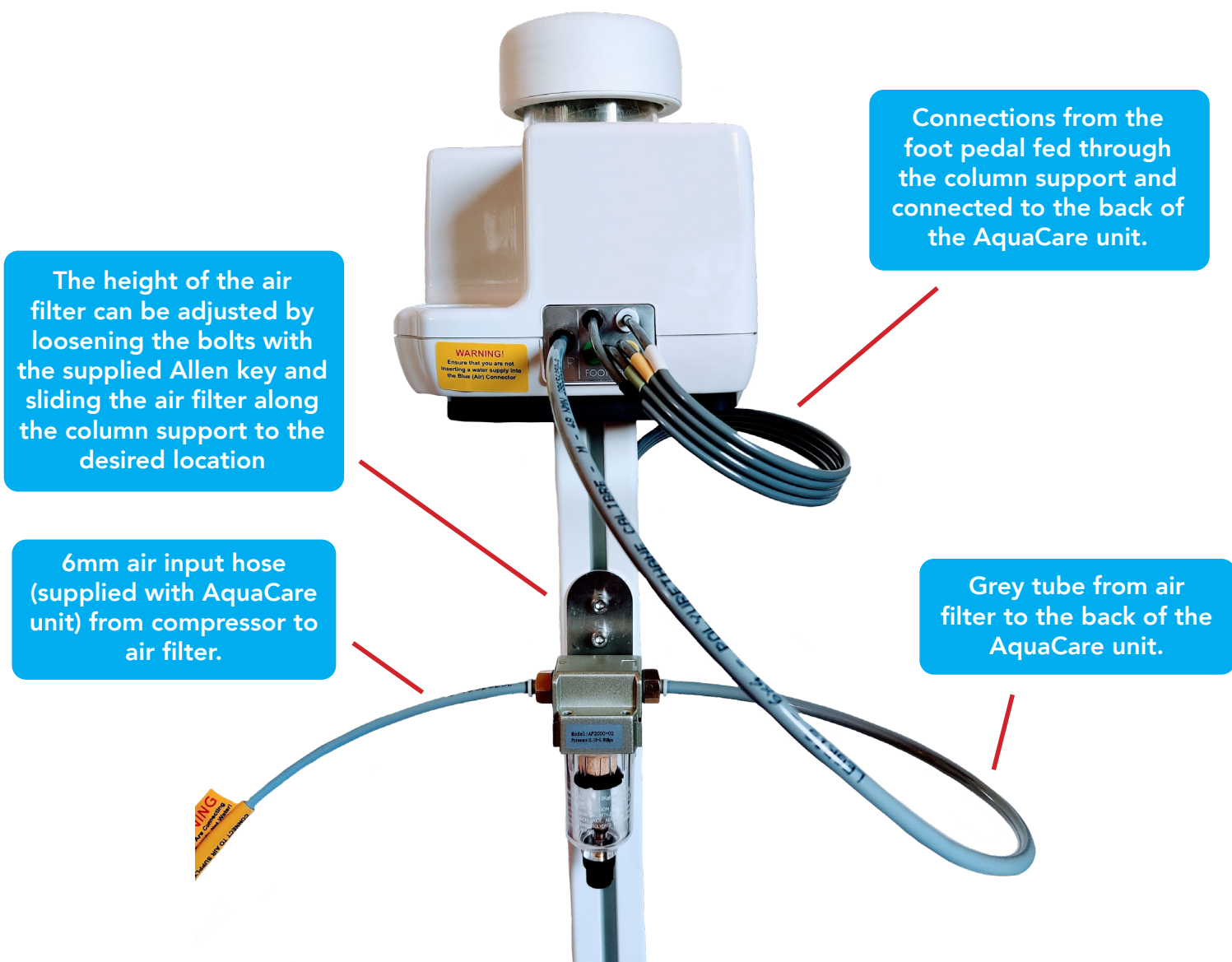

**NOTE:** To completely remove and replace the air filter, the column support must first be removed from the metal base. Loosen the screws of the air filter with the supplied allen key and slide the filter off the column support. Slide on a new air filter by aligning the T-Slot nuts with the column rail profile.

## Parts List Illustration

Profile View

- 1**

Column Support




- 2**

Metal Base Assembly


- 3**

Fixing Components  
(For Column Support)



Socket Head  
Screw



Metal Washer  
x 2



4mm, 5mm & 6mm  
Allen Keys
- 4**

Support Shelf Assembly  
(Only One Support Shelf Is Provided)


- 5**

Filter with Bracket Assembly


- 6**

T-Slot Nut and Screw  
(For Air Filter)



x 2



x 2
- 7**

Clear Tube  
(For DryAir Unit, If Applicable)


- 8**

Grey Tube



