

**Contents/Contenido**

SHOWN WITH OPTIONAL DAYLIGHT LOADER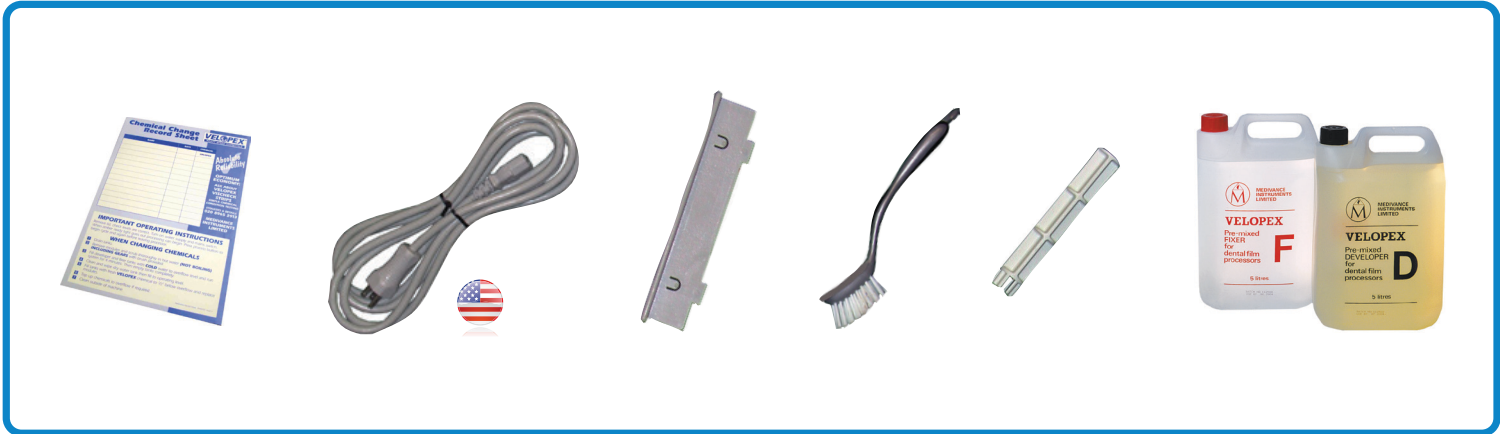

**Display/Pantalla**

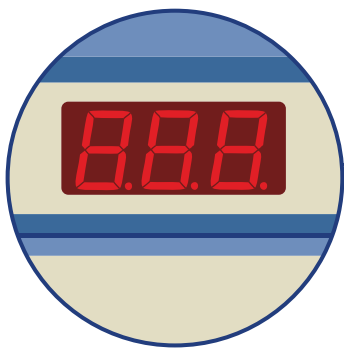- L 1 d**    **The lid is not closed correctly / La tapa no está cerrada correctamente**
- t L 0**    **The machine is warming up / La máquina se está calentando**
- E 2 1**    **Developer tank needs refilling / El depósito del Revelador debe ser rellenado**
- E 2 2**    **Fixer tank needs refilling / El depósito del Fijador debe ser rellenado**
- - -**    **Ready to use / Listo para usar**
- -**    **Cooling down / Enfríamiento**

## Installation/Instalación

**1**

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
11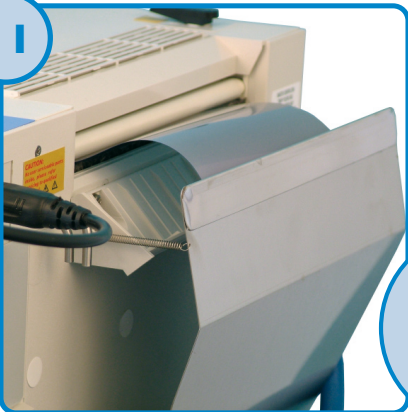


After 5 minutes the processed film will exit into the film collector / Tras 5 minutos, el carrete procesado saldrá del receptor del carrete

SHOWN WITH OPTIONAL DAYLIGHT LOADER

# Plumbing





## USA Plumbing Arrangements

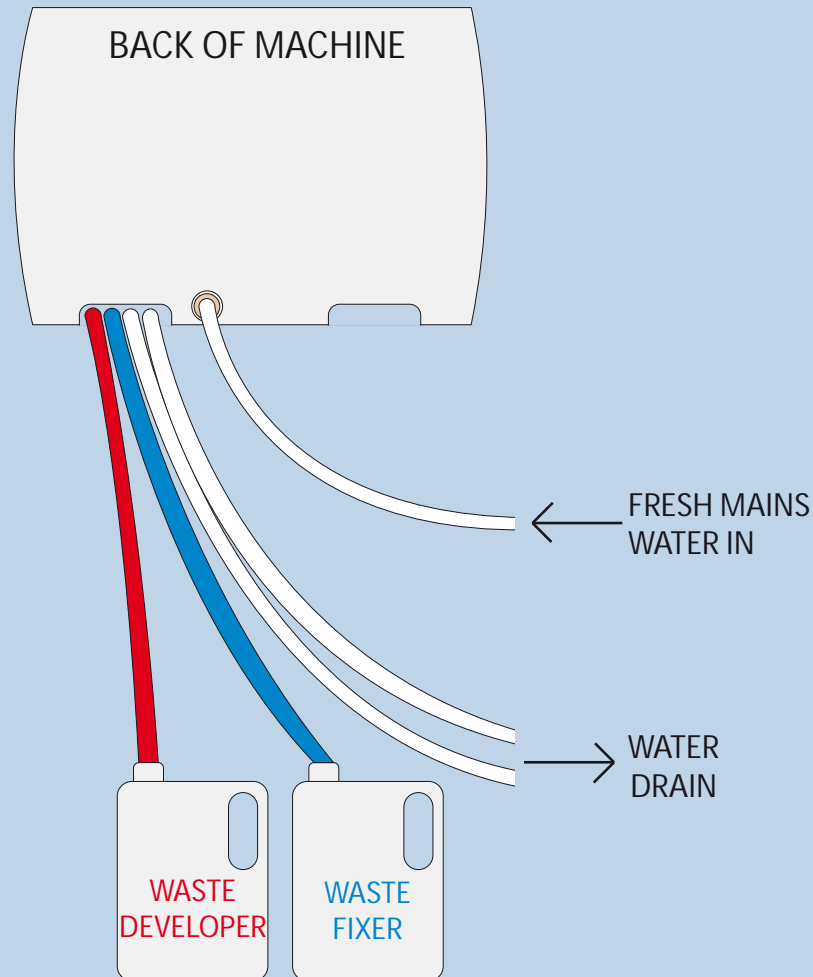**YOU WILL BE REQUIRED TO PROVIDE TWO CONTAINERS FOR WASTE CHEMICALS:  
ONE FOR DEVELOPER AND ONE FOR FIXER SOLUTION**

- **IMPORTANT** Do not mix waste Developer solution with Fixer solution as they react together and give off toxic gases.
- Dispose waste chemicals in line with local requirements.
- When emptying chemicals from the machine tanks ensure tanks are completely empty before refilling. The above is important to avoid contaminating the new chemical with residual old solution.
- Old chemicals will not fully empty from the machine tanks if the waste pipes are immersed in the waste chemicals in the containers.
- To ensure full draining lift the waste pipe clear of the waste chemical to allow it to fully empty into the container from the machine.
- Clean machine transport modules before returning them to their respective tanks.
- Do not mix up modules always ensure they go back into the tanks they came from to avoid contaminating the fresh chemical.

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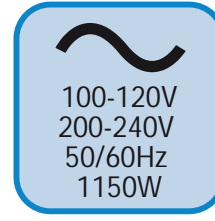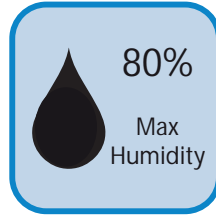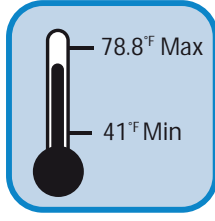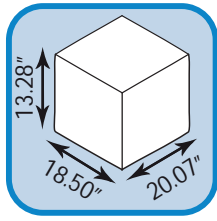Utilice piezas Velopex

For More Detail Refer To Manual

**OFFICE: USA**  
**VELOPEX INTERNATIONAL**  
**105 EAST 17TH STREET**  
**SAINT CLOUD**  
**FLORIDA 34769**  
**TEL: +1 (407) 957 3900**  
**FAX: +1 (407) 957 3927**  
**ENQUIRIES@VELOPEX.COM**

**WWW.VELOPEX.COM**

9051184

Extra Quick Start Guide Iss. 3  
I/LIT0150P Issued 10/25  
Velopex is a trademark of  
Medivance Instruments Limited.