



**Contents**

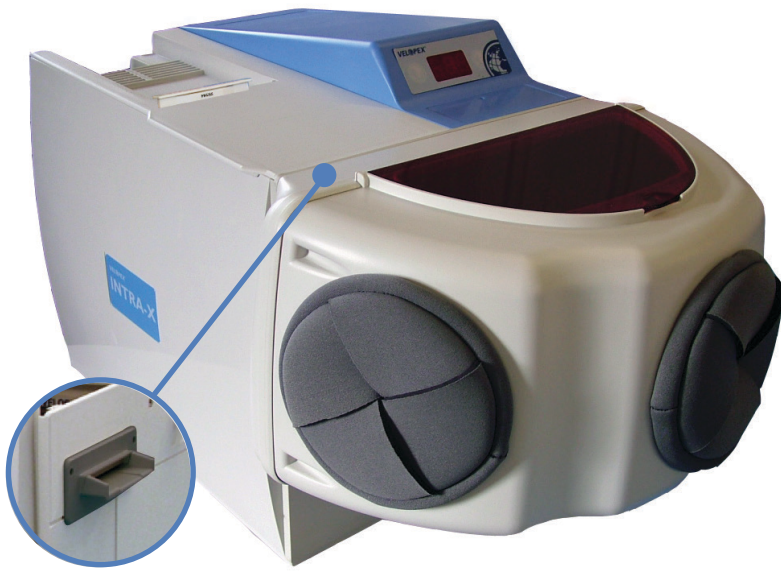

**Display**



<b>Lid</b>	The lid is not closed correctly
<b>tLo</b>	The machine is warming up
<b>---</b>	Ready to use
<b>- -</b>	Cooling down

# Installation

## Daylight Loader


A



Remove black stud covers from the front panel, and peel protective tape off the loader. Slide the loader over the film guide



B

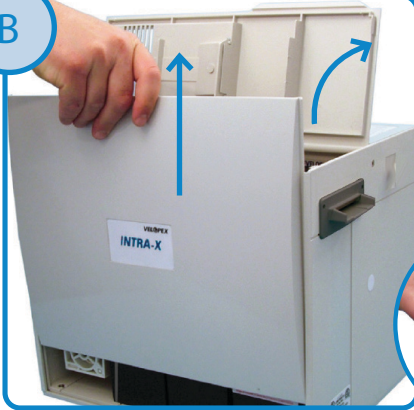


Lift the lid and remove the side panel. Slide out the modules and remove all the packaging




C



Thread the two retaining screws through the front panel. Inside the loader, screw the black knob onto the stud

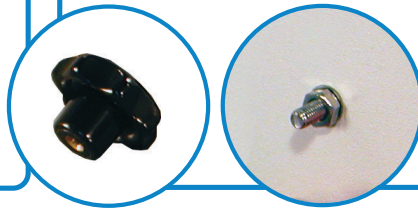

**Warning: Never mix the chemicals together**

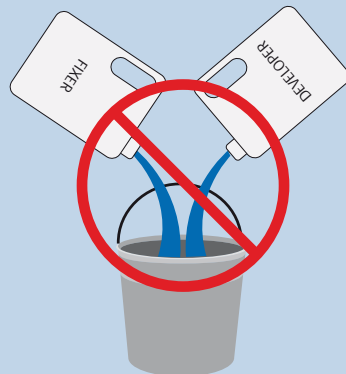


**Check with your local authority on how to dispose of chemical**




# Processor

1




**Fill up the tanks to the lowest edge of the tank window.**

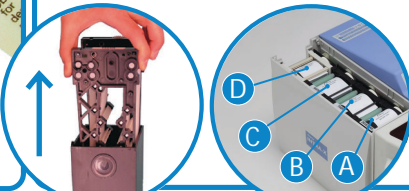


- A = Developer
- B = Fixer
- C = Water
- D = DO NOT FILL

2



**Apply silicon grease to the heaters**




3

(Drive Pin & Drive Dog)


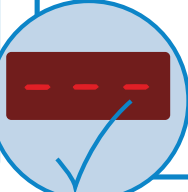


**Slide the tanks back into the machine. Make sure the drive pin on the module slides into the recess. Replace the side panel and lid**


4



**Plug in the machine using the electric cord and switch on. Wait until the "Ready" symbol is displayed**




5

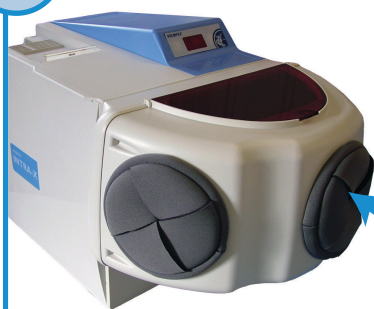


Put your hands through the loader gloves and place the film inside the loader

6

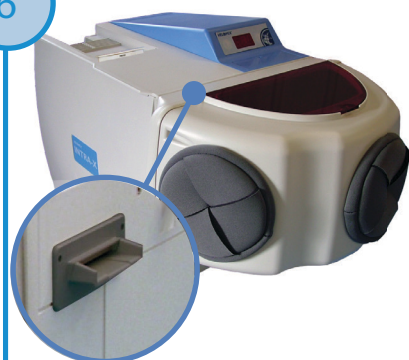

Feed the film squarely into the film slot. Do not remove hand from Daylight Loader until the film has completely entered the machine. Keep the lid CLOSED

7

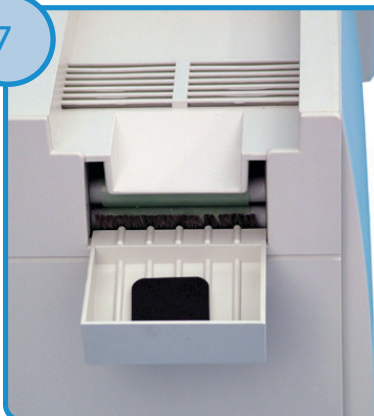


After 5 minutes the processed film will exit into the film collector



5 mins


8

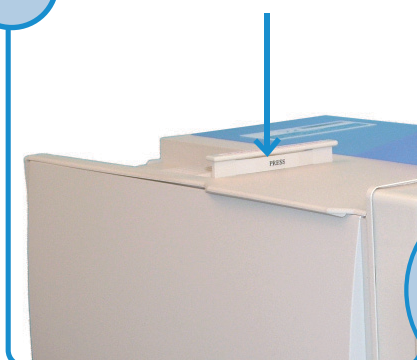

For quick viewing of wet films, press down the Endo slide. This will halve the processing time to 2.5 minutes






2.5 mins

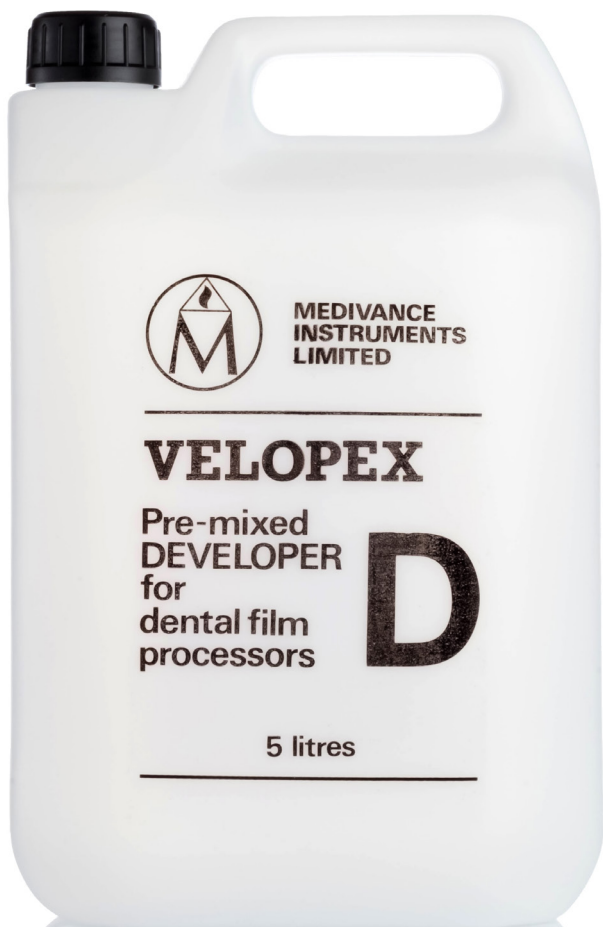


**VELOPEX**<sup>®</sup>

## Properties

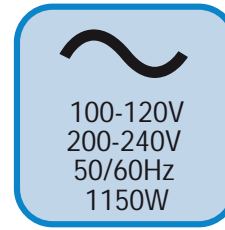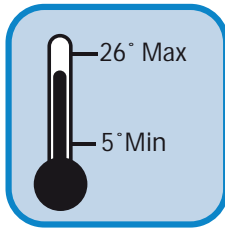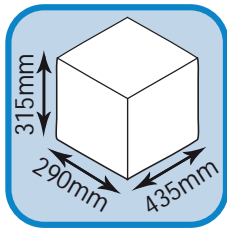

## Safety

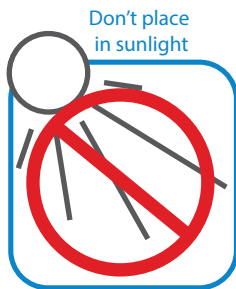

## Manual



**For further information on  
installation, operation, maintenance  
and service, see the manual**

OFFICE: UK  
Medivance Instruments Ltd  
Barretts Green Road  
Harlesden, London  
NW10 7AP  
Tel: +44 (0) 20 8965 2913  
Fax: +44 (0) 20 8963 1270  
enquiries@velopex.com

OFFICE: USA  
Velopex International  
105 East 17th Street  
Saint Cloud  
Florida 34769  
Tel: +1 (407) 957 3900  
Fax: +1 (407) 957 3927  
enquiries@velopex.com



Intra Quick Start Guide Iss. M  
I/LIT0140P Issued 10/2025  
Velopex is a trademark of  
Medivance Instruments Limited.



Hi-Tech  
Diagnostics  
Service